

# Vision Derma<sup>®</sup> Pro

Expert dermatoscopy examination  
for professionals





# Vision Derma® Pro

## Digital dermatoscopy



- 1 Main toolbar**  
 Basic working tools take minimum size to retain space for working with images.
- 2 Patient information**  
 All information about patients is presented in tabular form. Simple and convenient organization will help you not to waste time on searching.
- 3 Human body diagram**  
 When a dermatoscopy case is added, it is linked to a human body diagram considering patient's sex and body area.
- 4 Dermatoscopy cases**  
 Synchronous selection of dermatoscopy case in the table and the human body diagram.
- 5 Examination preview**  
 Thumbnail gallery for quick preview of a dermatoscopy examination images.
- 6 Dermatoscopy examinations**  
 All dermatoscopy examinations are presented in tabular form. Convenient sorting and filtering to find examinations.

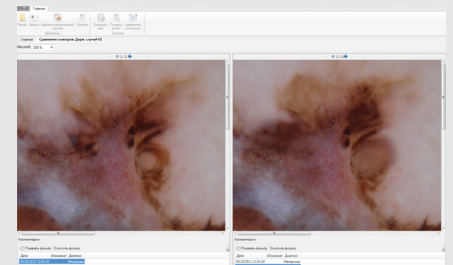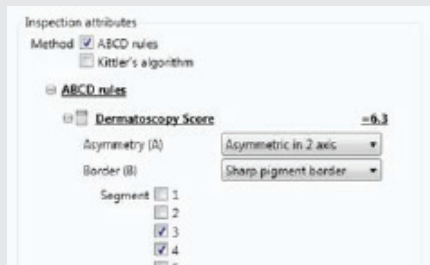

## Diagnostics



Choice of various algorithms and diagnostics methods, ABCD, Kittler, etc. Depending on the selected algorithm, the system will calculate the dermatoscopy index and propose you the review of a diagnose according to the criteria selected by you.

## Atlas

Develop your professional expertise by creating and maintaining a dermatoscopy atlas. Add the required sections and text comments to dermatoscopy images.

## Examination comparison

Assessment of change in a dermatoscopy case at different examinations. Use Zoom feature to make a more detailed study of a dermatoscopy image.

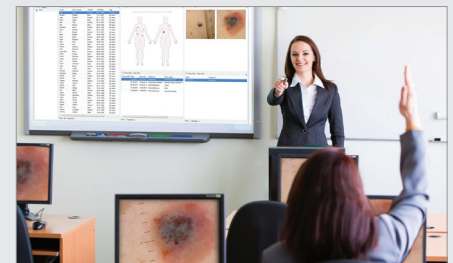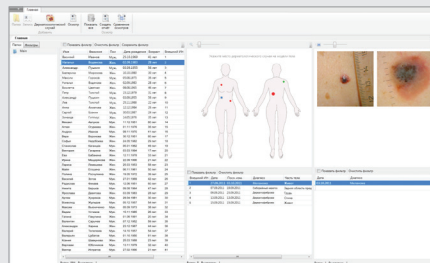

## Database for archive keeping

The system saves all data automatically, excluding any possibility of losing valuable information. Advanced data management tools, like filtering and sorting, will help you not to waste time on searching.

## Remote access and network capabilities

Data exchange with HIS. Storage of images and reports in the database allows the organization of an information network for the medical institution. Your colleagues can review case results, while at any working place.

## Education and professional development

By working with Vision Derma® Pro system, doctors improve their expertise every day, thanks to the use of atlas, review of images and comments as well as discussion with colleagues and experts.

## You decide how your report will look like

You can take into account different template requirements (form and content). Your report may take from 1/3 to 2-3 pages depending on conclusion details of the research carried out.

DERMATOSCOPY ANALYSIS REPORT		DermaCLINIC
Category	Medical examination	Examination date/time
Sample ID	113	Test number
Patient ID	123456	Examination
Name	John Smith	Address
Birth date	12/12/1975	Department
Gender	Female	Physician
Address	123 Street St	Physician name
Insurance	Private	Referral letter
Medical record	4567890123	
Comments		
A formation appeared six months ago after a summer holiday		
Dermatoscopy study		
		Dermatoscopy study ID: 18 Date: 10/11/2011 Last modified date: 10/11/2011 Study region: Melanoma region Diagnosis: Melanoma
Analysis results		
<b>Dermatoscopy effects</b> Structure: Asymmetry in both axes Border: Irregular Segment 1: Regular Segment 2: Regular Segment 3: Irregular Segment 4: Irregular Segment 5: Regular Segment 6: Regular Segment 7: Regular Segment 8: Regular Color: Brown, red, yellow, dark brown, greyish blue Dermatoscopy structures: Structureless areas, dots, globules Other: None Size: More than 5 mm Dermatoscopy results: 9.8 Score: 9.8		<b>Dermatoscopy effects</b> Asymmetry in both axes Border: Irregular Segment 1: Regular Segment 2: Regular Segment 3: Irregular Segment 4: Irregular Segment 5: Regular Segment 6: Regular Segment 7: Regular Segment 8: Regular Color: Brown, red, yellow, dark brown, greyish blue Dermatoscopy structures: Structureless areas, dots, globules Other: None Size: More than 5 mm Dermatoscopy results: 9.8 Score: 9.8

Images	
1. Clinical image      2. Dermatoscopy	
Recommendation	
Consult with oncological surgeon	
Diagnosis	
Melanoma	
Treated by	
Name: Dr. Chris Rogers Position: Dermatologist	Date: 10/10/2011 Time: 11:18
Signature	

## Vision Derma® Pro / Set 1



### Set for digital dermatoscopy

#### Includes:

- DermLite DL3N dermatoscope (3Gen, USA)
- Canon PowerShot G digital camera with photoadapter
- Wi-Fi kit for wireless transfer of dermatoscopy images
- Vision Derma® Pro software

## Vision Derma® Pro / Set 2



### Set for digital dermatoscopy

#### Includes:

- DELTA 20° Plus dermatoscope with photo accessories (Heine, Germany)
- Canon DSLR camera with macro lens for clinical imaging
- Wi-Fi kit for wireless transfer of dermatoscopy images
- Vision Derma® Pro software

## Vision Derma® Pro / Set 3



### Set for digital dermatoscopy

#### Includes:

- Ri-Derma® dermatoscope (Riester, Germany)
- Digital camera with photoadapter
- Wi-Fi kit for wireless transfer of dermatoscopy images
- Vision Derma® Pro software

## Vision Derma® Pro with your dermatoscope



### Set for digital dermatoscopy

#### Includes:

- Your dermatoscope with photo accessories
- Wi-Fi kit for wireless transfer of dermatoscopy images
- Vision Derma® Pro software

We reserve the right to change the specification without notice.

