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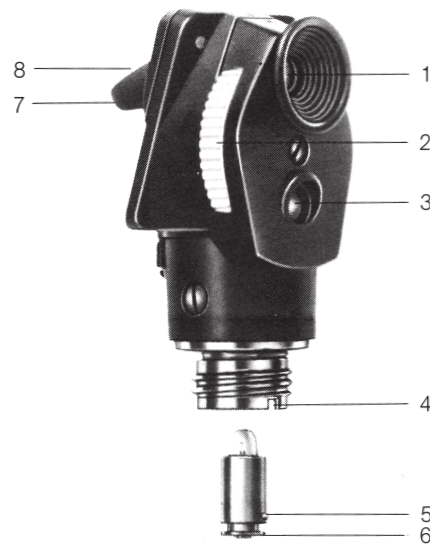
HEINE

HEINE Optotechnik GmbH & Co. KG
Kientalstr. 7 · 82211 Herrsching · Germany
Tel. + 49 (0) 81 52 / 38 - 0 · Fax + 49 (0) 81 52 / 38 - 202
E-mail: info@heine.com · www.heine.com
tec 8306 1/1.07

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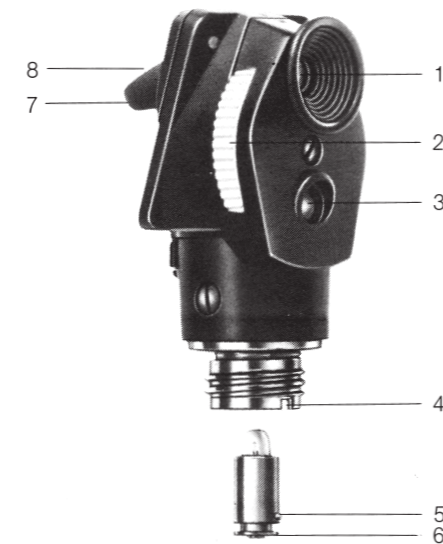
Warranty

We guarantee this device (except bulbs) for two years from date of purchase. We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product, the buyer must prove that any such defect was present at the time the product was delivered. This guarantee excludes any defects which occur because of wear and tear, incorrect use, the use of non-original HEINE parts, (especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance). Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

Bought from

date

Instructions

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Read these instructions carefully before using the HEINE mini Keyhole loupe and keep them in a safe place for future reference.

In accordance with local regulations this product should be disposed of as an electronic device separately.

HEINE mini 2000® Battery Handle

Batteries:
2 alkaline cells (AA/ LR 6)

Instrument connector:
Internal thread in handle head (1).

Switching on:
Push switch (2) down – a red "on" indicator comes into view.

Switching off:
Push switch (2) up. If the handle is put back firmly into a coat pocket the handle is automatically switched off.

Changing the batteries:
Unscrew the end-cap (3) and shake out the old batteries. Insert new batteries with the plus contact facing the handle head.

Cleaning:
Wipe with a soapy cloth or a little alcohol.

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The mini Keyhole loupe is used to inspect small bores and cavities of 0,3 mm or more. It can also be used as a focusable viewer for objects at a distance from 5 to 250 mm by means of a range of 17 lenses of different powers.

Instructions for use

- Attach the keyhole loupe to the mini 2000 handle by the screw thread.
- Hold the instrument with thumb and forefinger so that the on/off switch and the lens wheel (2) can be operated comfortably.
- Switch on and hold the mini keyhole loupe as close as possible to your eye.
- Look through the eyepiece (1) at the object and focus by rotating the lens wheel with the forefinger.

When examining very small objects it may be easier to aim the light beam and rest the positioning guide (7) on it. The bevelled end of the guide facilitates this.

If the distance to the object is known you can adjust the lens wheel before the inspection. The values shown in window (3) are average viewing distances for the selected lenses.

Changing the bulb

- Unscrew the instrument from the handle.
- Grasp the bulb by the collar (6) and pull it out.
- Wipe the glass of the new bulb with a soft cloth and push in the new bulb so that the locating pin (5) fits the groove (4).

Use only HEINE XHL® Xenon halogen bulbs **T-01.88.042**.

Please note: We cannot guarantee the performance of the HEINE mini keyhole loupe if other bulbs are used.

Cleaning

Wipe clean with soap solution or an alcohol-based cleaner which is recommended for use on plastic. Use a cotton bud or a micro-fiber cloth to clean the glass surface.

Accessories and Spare Parts

Keyhole loupe with 3,5V bulb, without handle	T-01.24.100
HEINE mini 2000® battery handle (with batteries)	T-01.79.009
HEINE XHL® Xenon halogen bulb 2,5 V	T-01.88.042